



# COVID-19 (SARS-CoV-2) Antigen Test Kit (Colloidal Gold)



ANHUI DEEPBLUE MEDICAL TECHNOLOGY CO.,LTD

COVID-19 (SARS-CoV-2) Antigen test kit (Colloidal Gold) is used for in vitro qualitative detection of the antigen of SARS-CoV-2 in human Nasal Swab, Nasopharyngeal swab, Oropharyngeal swab by double antibody-sandwich method and colloidal gold immunochromatography. Double antibody-sandwich method can ensure a low antibody cross reaction, effectively improve the sensitivity and specificity of the product and reduce false positive. It can be used for early screening of infection and is an effective supplement to nucleic acid (PCR) detection.

Website: [www.dbluemedical.com](http://www.dbluemedical.com)

Contact: 0551-65326797

Address: 4 floor, 1#, D zone, Pearl Industrial Park, 106 innovation Avenue, high-tech Zone Hefei, Anhui, China, 230088

## Products information

Name: COVID-19 (SARS-CoV-2) Antigen Test Kit (Colloidal Gold)


Specification: 1 pcs/box, 10 pcs/boc, 25 pcs/box

Specimen: Nasal Swab, Nasopharyngeal swab, Oropharyngeal swab

Storage: 4-30°C

## Test Procedure


### 1. Specimen Collection (Three Options)

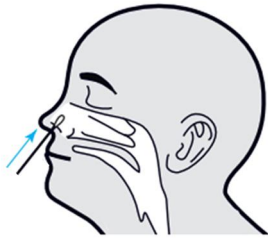

Nasal Swab



Nasopharyngeal swab

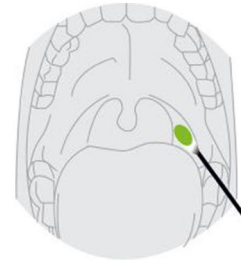


Oropharyngeal swab

### 2. Specimen Preparation



### 3. Testing


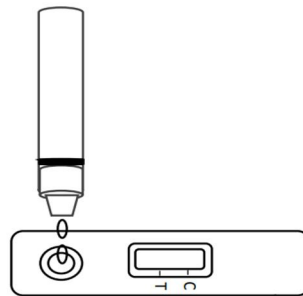


Hold the extraction tube vertically and add two drops of the test specimens into the specimen well (s). Start the timer.

### 4. Interpretation of test results



20 minutes

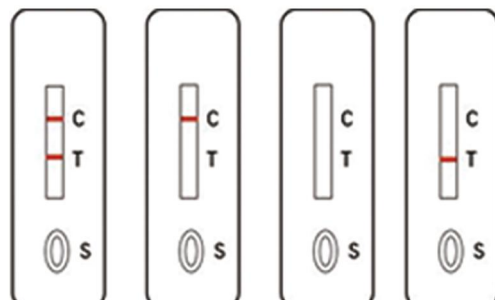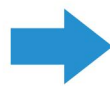

Positive

Negative

Invalid

Invalid

## Product Features

- No pre-processing required, simple operation, do not need any auxiliary equipment.
- Rapid test, easy to interpret the results in 20 minutes.
- The effective supplement of the COVID-19 Antibody test window period.
- Transported and stored at room temperature.